

IV3000 Application

IV3000 10cm x 12cm

Suitable for dressing central line catheters


Step A



Clean and thoroughly dry the application area in accordance with local protocols, which may involve sutures.

First remove section 1 from the end of the dressing. This contains two sterile strips and a date label.

Step B

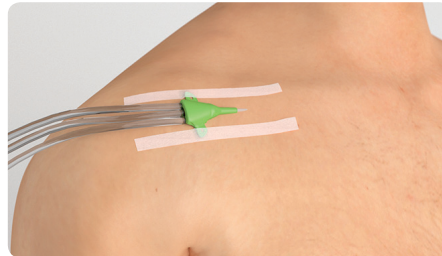


The sterile strips can be used on the catheter wings or body for improved security, to secure tubing or to stabilize catheter lumens.

Step C



Remove the backing paper marked 2.

Step D

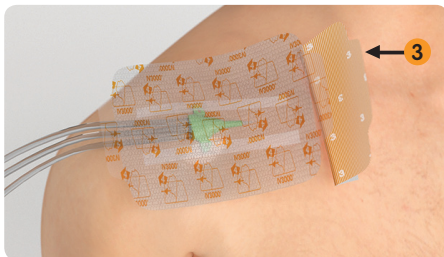


Centre the dressing over the catheter insertion site.

Place the dressing on the catheter hub and adhere the dressing to the patient's skin.


Remove the backing paper marked 3.

Step E

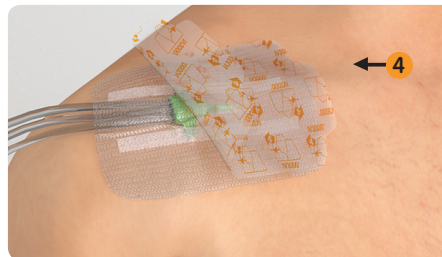


Smooth the edges of the dressing.
Using the tab, gently peel off the top backing paper marked 4.

Step F

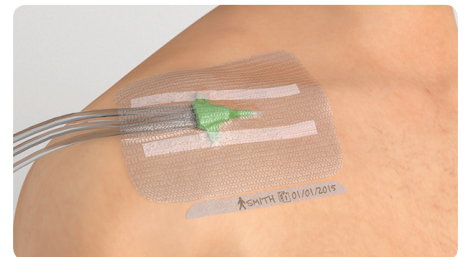

Mould the film around the catheter, ensuring that the dressing is secure and the patient is comfortable.

Finally, after recording relevant information on the date label, place the date label from section 1 on the skin or the dressing (without obscuring the view of the catheter site).

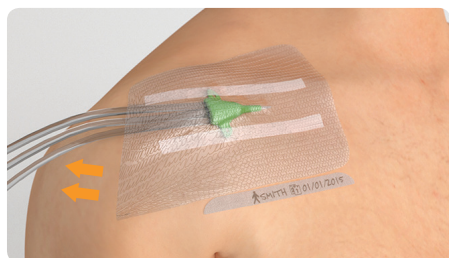

Removal

Take a corner of the film and stretch it parallel to the patient's skin while stabilizing the patient's skin and catheter with the other hand.

Release and repeat the process.

Ordering codes

Dressing						
Size	5cm x 6cm	6cm x 7cm	7cm x 9cm	9cm x 12cm	10cm x 12cm	11cm x 14cm
S&N Code	66004011	4007	4006	66004009	4008	66800512
Carton	100	100	100	50	50	25

Wound Management
Smith & Nephew
Medical Ltd
101 Hessele Road
Hull HU3 2BN
UK

www.smith-nephew.com/wound
www.iv3000.com

[®]Trademark of Smith & Nephew
© Smith & Nephew March 2015
51511

T +44(0) 1482 225181
F +44 (0) 1482 328326