

IV3000 Application

IV3000 Frame Delivery Dressing 6cm x 7cm

Suitable for dressing peripheral catheters

FRAME DELIVERY


Step 1



Clean and thoroughly dry the application area in accordance with clinical protocols.


The sterile strips supplied on the backing paper can be used on the catheter wings or body for improved security, to secure tubing or to stabilise catheter lumens.

Step 2

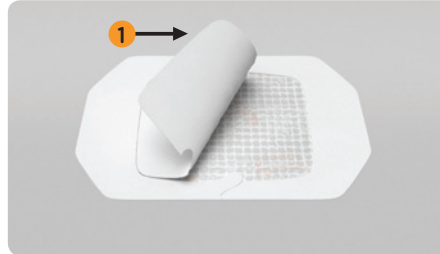


Remove the central window marked **1** at scored area

Step 3

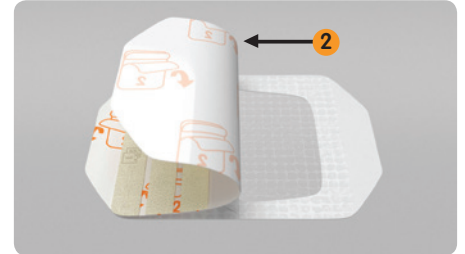

Remove the printed backing paper marked **2** from the dressing

Step 4

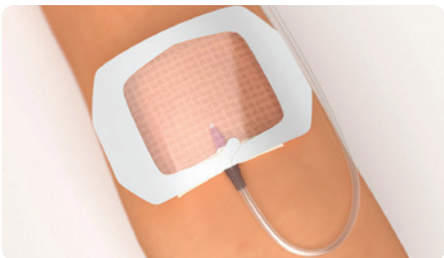

Holding the frame, centre the dressing over the catheter insertion site.

Note you have full visibility of the insertion site for a more accurate application.

Step 5



Place the dressing on the catheter hub and adhere the dressing to the patient's skin. Mould the film around the catheter.

Remove the paper frame using the tab.

Step 6



Smooth down the edges of the dressing, ensuring the dressing is secure and the patient is comfortable.

Finally after recording relevant information on the date label, place it on the patient's skin or the dressing (without obscuring the view of the catheter site).



Removal

Take a corner of the film and stretch it parallel to the patient's skin while stabilising the patient's skin and catheter with the other hand.

Release the film and repeat the process.

Ordering codes

Dressing		
Size	6cm x 7cm	10cm x 12cm
S&N Code	59410082	59410882
Carton	100	50

Wound Management
Smith & Nephew
Medical Ltd
101 Hessle Road
Hull HU3 2BN
UK

www.smith-nephew.com/wound
www.iv3000.com

[®]Trademark of Smith & Nephew
© Smith & Nephew September 2015
51515

T +44(0) 1482 225181
F +44 (0) 1482 328326