

IV3000 Application

IV3000 Ported dressing 5cm x 6cm

Suitable for dressing peripheral catheters

Step A




Clean and thoroughly dry the application area in accordance with local protocols.

First remove section **1** from the rounded end of the dressing.


This contains two strips and a date label.

Step B

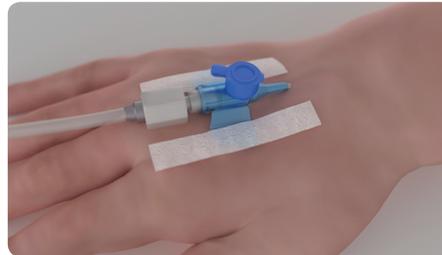

The sterile strips can be used on the catheter wings or body for improved security, to secure tubing or to stabilize catheter lumens.

Step C

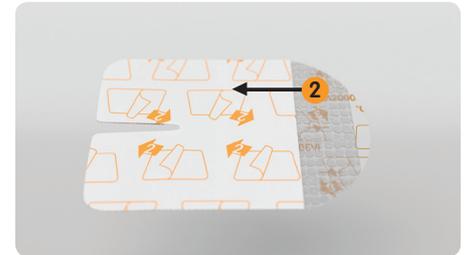

Remove the backing paper marked **2**.


Step D



Position the dressing over the catheter port

Smooth the edges of the dressing.

Step E

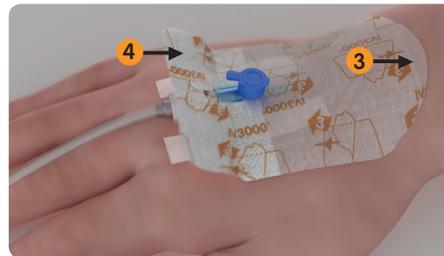


Using the tab, gently peel off the top backing paper marked **3**.

Remove the backing papers marked **4** from the non-woven sections and adhere over the catheter wings.


The non-woven sections can be gently pulled together and overlapped to ensure the catheter is well secured.

Step F

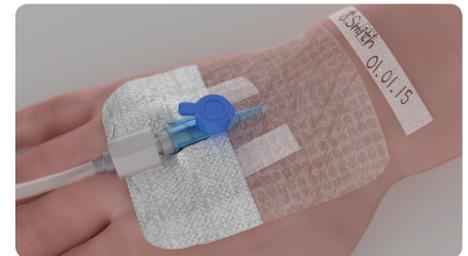

Mould the film around the catheter, ensuring that the dressing is secure and the patient is comfortable

Finally, after recording relevant information on the date label, place the date label from section **1** on the skin or the dressing (without obscuring the view of the catheter site).

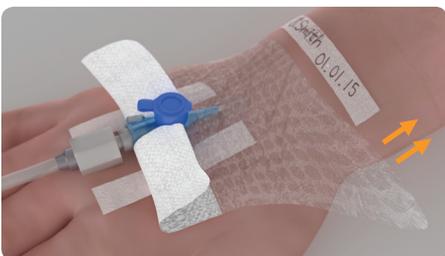




Removal

Gently peel back the non-woven wings to the point at which they reach the film section.

Then take a corner of the film and stretch it parallel to the patient's skin while stabilizing the patient's skin and catheter with the other hand.

Release and repeat the process

Ordering codes

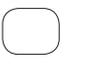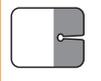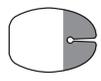Dressing						
Size	5cm x 6cm	6cm x 7cm	7cm x 9cm	9cm x 12cm	10cm x 12cm	11cm x 14cm
S&N Code	66004011	4007	4006	66004009	4008	66800512
Carton	100	100	100	50	50	25

Wound Management
Smith & Nephew
Healthcare Ltd
Healthcare House
101 Hessele Road
Hull HU3 2BN

T 01482 222200
F 01482 222211

www.smith-nephew.com/wound
www.iv3000.com

[®]Trademark of Smith & Nephew
© Smith & Nephew September 2014
50988