

# IV3000 Application

IV3000 Ported 7cm x 9cm

Suitable for dressing peripheral catheters

## Step A




Clean and thoroughly dry the application area.

First remove section **1** from the rounded end of the dressing.


This contains two strips and a date label.

## Step B

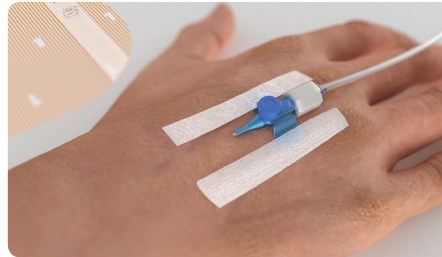

The strips can be used on the catheter wings or body for improved security, to secure tubing or to stabilise catheter lumens.

## Step C

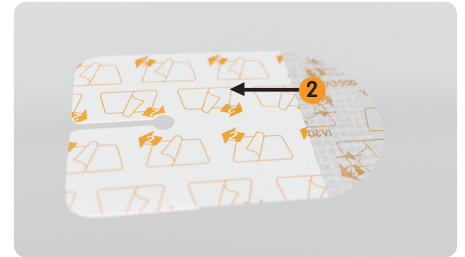

Remove the backing paper marked **2**.


## Step D



Position the dressing over the catheter port.

Smooth the edges of the dressing.

## Step E

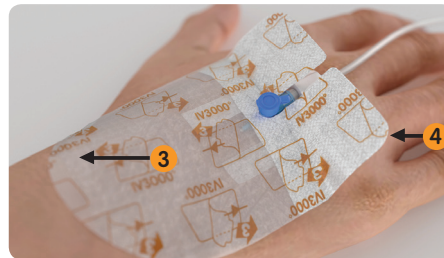

Using the tab, gently peel off the top backing paper marked **3**.

Remove the backing papers marked **4** from the non-woven sections and adhere over the catheter wings.

## Step F



Mould the film around the catheter, ensuring that the dressing is secure and the patient is comfortable.


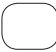




Finally place the date label from section **1** on the skin or the dressing (without obstructing the view of the catheter site).



## Removal

Gently peel back the non-woven wings to the point at which they reach the film section. Then take one corner of the film and stretch it parallel to the patient's skin while stabilising the patient's skin and catheter with the other hand. Release and repeat the process.

## Ordering codes

Dressing						
Size	5cm x 6cm	6cm x 7cm	7cm x 9cm	9cm x 12cm	10cm x 12cm	11cm x 14cm
S&N Code	66004011	4007	4006	66004009	4008	66800512
Carton	100	100	100	50	50	25

### Wound Management

Smith & Nephew  
Healthcare Ltd  
Healthcare House  
101 Hessele Road  
Hull HU3 2BN

T 01482 222200  
F 01482 222211

[www.smith-nephew.com/wound](http://www.smith-nephew.com/wound)  
[www.iv3000.com](http://www.iv3000.com)

\*Trademark of Smith & Nephew  
© Smith & Nephew June 2014  
50984